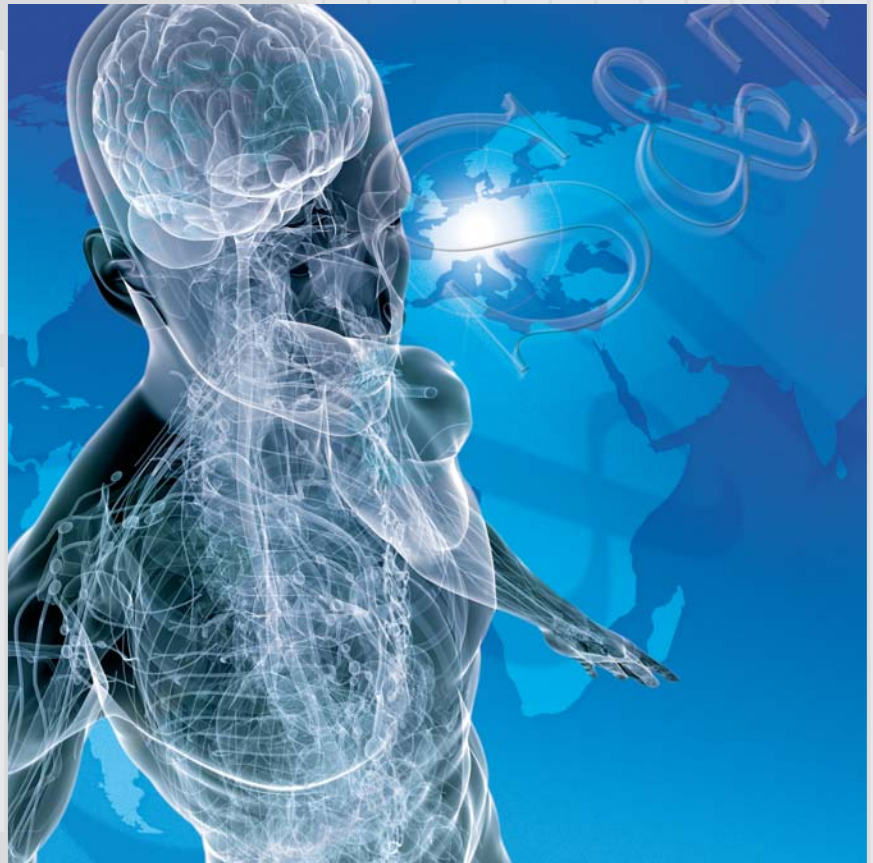
World of Microsurgery
Welt der Mikrochirurgie

S&T[®]

S&T[®]
Confidence
Vertrauen

All
S&T[®] instruments
carry a lifetime
warranty against
defects in
workmanship
and materials.

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S&T[®]

All **S&T**[®] products must be used exclusively for their intended purpose.

All instruments are subject to technical changes in design and construction. We will supply the current model.

Our general terms and conditions of sale and deliveries are applicable.

Reprints of this catalogue in whole or in part is not permitted. Printed or electronic matters of any kind for which illustrations of our products are required with or without the respective article number are subject to our written consent.

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Basic informations Grund- Information

Needleholders

There is more to a good needleholder than meets the eye.

This is especially true for S&T®'s line:

- **Secure grip** on even the smallest needle and finest suture material is possible due to parallel closure over the entire jaw length.
- **Easy tying** of the finest suture material is possible due to perfectly designed jaws.
- Closing pressure is precisely calibrated to facilitate holding of the instrument while **minimizing hand fatigue**.
- **Safe suture handling** as all sharp corners and rough spots are eliminated.

Selection of the proper Needleholder

In addition to the observations described in the beginning pages of the catalog, there are four points that you should take into consideration when selecting a needleholder:

- your preference for the needleholder tip, straight or curved
- your preference for locking or non-locking needleholder
- your preference for a hinged or non-hinged needleholder
- Round or flat handle

Needleholder tip

Curved tip needleholders are the most commonly used instruments, however on occasion some surgeons prefer straight needleholders.

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Basic informations

Grund-Information

Needleholders with the designation "C" in the article code are based on the design by Castroviejo and are basically fine, flat handle needleholders. The designation "B" stands for round handle needleholders which were introduced by Barraquer.

Both designs originate from ophthalmic instrument designs.

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Prof. Dr. Ignacio Barraquer y Barraquer, Barcelona, Spanien

Needleholders with lock

The lock on a needleholder is mainly intended to be a convenient means for parking the needle and passing it to the surgeon. The lock is not intended to be an aid for holding the needle while suturing.

S&T® needleholders with a lock are carefully designed and calibrated. The lock may be disengaged and re-engaged without the inconvenience of letting go of the needle. Engagement and disengagement is marked by an audible click.

Since the finger pressure required to release the lock may cause unwanted movement at a critical moment, we recommend that the passing of the needle through tissue should be done with the lock not engaged.

Important!

Refrain from using microsurgical needle holders to hold a macrosurgical needle. Needles over 8 mm length are so thick and the force required to grip and pass them is so high, that even a single use may permanently damage the jaws of a micro needleholder.

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
Angulated forceps are extremely useful light-duty needle holders, very widely preferred for lab work. They are also useful for fine clinical work on vessels up to 2 mm in diameter. Above this size a hinged needle holder is recommended, since a firmer grip on the needle is required when working with thicker tissue.

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Needleholder forceps have stronger tips than standard microsurgical forceps. Needleholder forceps are produced to the same rigid specifications of our spring-handled needleholders. **S&T** needleholder forceps are designed to facilitate rotational technique.

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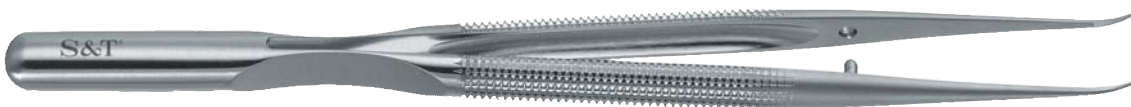NHF-15 RM-8 (1:1)

6" <i>150 mm</i>	8 mm	0.6 mm	Balanced Line	NHF-15 RM-8	00167
7 1/8" <i>180 mm</i>	8 mm	0.6 mm	Balanced Line	NHF-18 RM-8	00176

SUPERFINE

Delicate tips and blades
Feine Spitzen und Klingen

Needleholder <i>Nadelhalter</i>	Double Action <i>Doppelgelenk</i>	Round handle, curved <i>Rundgriff, gebogen</i>
---	---	--

Double action needleholders are especially advantageous in situations where access to an operation site is limited. Double action needleholders feature a reduced handle width compared to a conventional spring handled needleholder.

These instruments are an original **S&T**[®] development.

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Instrument cases

Instrumentendosen

Special Instruments
and Accessories

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S&T[®]

S&T AG · Zollstr. 91 · CH-8212 Neuhausen, Switzerland
Fon: +41 (0) 52 672 74 07 · Fax: +41 (0) 52 672 74 02
web: www.microsurgery.ch · e-mail: info@s-and-t.net

Distributed by:

All products are subject to technical changes. © 04.2009 S&T AG · CH-8212 Neuhausen, Switzerland